TOTAL RESPIRATORY

RSG Safety 300 Series Half Masks

NEOPRENE RUBBER: 30 02 01 (S, M, L, Black)

SILICONE: 30 01 01 (S, M, L, Blue)

**HALF MASK WITH
BAYONET SOCKET**

INSTRUCTIONS FOR USE

GB

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D

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NL

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FR

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ES

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IT

BRUGSANVISNING

DK

BRUKSANVISNING

S

BRUKSANVISNING

N

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FIN

Marinus Dammeweg 38 5928 PW Venlo The Netherlands

T: +31(0) 85 487 0395

E: sales@rsgsafety.com

www.rsgsafety.com

© copyright
2018

RSG SAFETY B.V. 2018 (GB)

1 (GB)**INSTRUCTIONS FOR USE**

RSG 300 SERIES HALF MASKS
30 02 01 • NEOPRENE RUBBER • S-M-L
30 01 01 • SILICONE • S-M-L

CONTENT:	PAGE
Important instructions	1
General	2
Application / Limitation	3
Fitting of RSG filters	4
Putting on - Test	5
Cleaning - Disinfection - Drying - Storage	6
Maintenance - Sub-component numbers - Approved institution	7
Tables	8
Marking	9

IMPORTANT INSTRUCTIONS:

In order to achieve correct and effective use of RSG HALF MASK, it is extremely important to read and comply with these instructions.

Only original RSG-spare parts may be used when maintaining and repairing RSG HALF MASK.

RSG SAFETY B.V. recommends that regular service and repair should be carried out by trained personnel. More extensive repairs should be carried out by RSG SAFETY B.V.' technical department.

RSG SAFETY B.V. abdicates all responsibility for repairs not carried out by RSG SAFETY B.V.' technical department.

GENERAL

(GB) 2

RSG HALF MASK is a CE-marked half mask, tested and approved in accordance with EN140:1998.

RSG HALF MASK covers the nose, mouth and chin. When an RSG FILTER is fitted to the RSG HALF MASK, the result is a filtration respirator which protects the user's respiratory organs against harmful gases, fumes and/or particles.

RSG HALF MASK is produced in **2 material types**:

RSG Neoprene Rubber is a soft material which is kind to the skin and is suitable for the majority of working situations in which a half mask is required.

RSG Silicone is kind to the skin, and is a very soft and comfortable material which is only affected by ageing to an extremely limited degree. Perfect for tasks where there is, for instance, a risk of adhesion of dyes.

RSG HALF MASK is produced for each material type in 3 sizes (Small-Medium-Large). The user of RSG HALF MASK can thus choose a mask which fits comfortably, tightly and safely to the face, irrespective of facial shape and size.

RSG HALF MASK is equipped with adjustable straps ("quick release") which ensure a rapid, perfect and comfortable fit.

The straps are fixed to a plastic front plate so that, when in use, the mask is pressed against the face with a uniformly distributed pressure, which provides the best and safest fit.

RSG SAFETY B.V. 2018 (GB)

3 (GB)**APPLICATION / LIMITATION**

If the polluted air's oxygen content is 17-21% (volume), RSG HALF MASK can be used with an RSG gas, RSG particle or RSG combined filter. Use an oxygen meter to ascertain the oxygen content of the air. Only RSG filters are tested and approved for use with the RSG HALF MASK. Follow the filter's instructions for use carefully.

If the oxygen content of the air is less than 17% (volume), an air-supplied respirator which is independent of the surrounding air must be used.

If the gas content of the air exceeds 0.1% (volume), or the air contains particularly noxious particles or gases, a full face mask should be used (RSG FULL FACE MASK).

Note that eye and head protection may also be necessary.

In areas where the use of respiratory equipment is necessary, there is often a danger of explosion. If this is the case, then the relevant precautions must be observed.

If the user has certain physical features (such as a beard or thick sideboards), then the RSG HALF MASK might not give a satisfactorily tight fit to the face. This will also apply if spectacle frames obstruct the facial seal. In such cases, the user carries the risk of damage to his health.

Leave the area immediately, check the mask and replace parts if:

1. The mask, parts thereof, or filters are damaged.
2. Breathing difficulties occur.
3. Dizziness or indisposition occurs.
4. Pollution can be tasted or smelt, or other irritation occurs.


.RSG HALF MASKS must never be modified or altered

FITTING OF RSG FILTERS


(GB) 4

When fitting RSG filters, two identical filters, suitable for the purpose for which they are to be used, should be fitted before entering the polluted area. When replacing filters, both RSG filters must be replaced at the same time.


1. At the bottom of the RSG filter, a "marking" can be seen. Hold the filter such that the index finger is located opposite the "marking".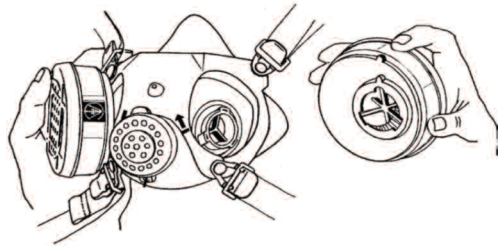

2. Guide the bottom of the RSG filter down onto the filter holder. When the index finger/"marking" is in line with the "arrow" on the front part of the half mask, then the filter socket and filter holder are in the correct position.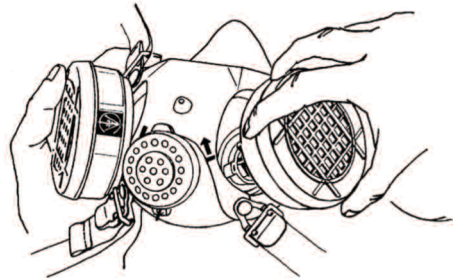

3. Turn the RSG filter in the direction of the arrow until it cannot be turned further. The filter is then correctly fitted. The same procedure should be followed when fitting the second filter.

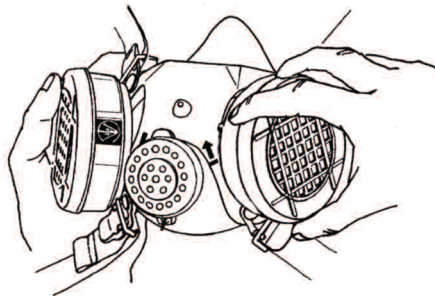RSG SAFETY B.V. 2018 (GB)

5 (GB)**PUTTING ON - TEST****PUTTING ON THE RSG HALF MASK:**

- 1.** With one hand, hold the mask in front of the face and place it so that the mouth, nose and chin are covered. Then pull the head straps over the head, and place the two-piece headband behind the head.
- 2.** Then take the bottom straps in both hands, place them behind the neck and fasten them together.
- 3.** Pull the head straps and the bottom straps tight. The impermeability of the mask can now be checked before entering the working area.

**CHECK BEFORE USE:**

RSG HALF MASK must always be checked before use in order to ensure that the half mask is not damaged and is clean. It is extremely important that a mask leakage test (see below) is always carried out before use.

**MASK LEAKAGE TEST:**

Place the palms of your hand over the open filter housings (plastic film or pieces of cardboard can also be used to cover the openings if your palms are not sufficiently large). Gently breath in and hold your breath for about 10 seconds. If the mask is sucked in towards the face, then it is correctly adjusted. If any leakage can be felt, the straps should be adjusted in order to stop the leak. If a correct fit cannot be achieved, then a detailed examination of the mask's individual components is necessary. Note that the size of the mask can be highly significant with regard to achieving the correct fit.

CLEANING-DISINFECTING-DRYING-STORAGE (GB) 6

CLEANING:

It is recommended that the mask be cleaned after use. Clean the mask and the other components (except the filters), in lukewarm water to which a mild detergent has been added (e.g., washing-up liquid or a liquid washing machine detergent) which does not contain perborate. If the mask is very dirty, a brush can be used to clean it. Do not use solvents (e.g., benzine or spirit) or very warm water (max. 60°C). The mask should be rinsed thoroughly in lukewarm running water (refer to cleaning intervals in table 3, page 8).

DISINFECTION:

Disinfect the mask components in a suitable solution after use. Use a disinfectant such as "MULTIDOR". Follow the instructions carefully (refer to disinfection intervals in table 3, page 8).

DRYING:

RSG HALF MASK should be air-dried or dried in a drying cabinet at a temperature not exceeding 60°C. Avoid direct sunlight.

STORAGE:

Store RSG HALF MASK in airtight and light-proof packaging in a cool and dry place.

NOTE!

Silicone has a tendency to absorb detergents and disinfectants. Irritation of the skin can occur as a result.

Therefore:

1. Reduce the time the substances are in contact with the mask, which is stated in the instructions, to a minimum.
2. Rinse thoroughly in running water for 10 minutes, after cleaning and disinfection of RSG HALF MASK 30 01 01.

RSG SAFETY B.V. 2018 (GB)

7 (GB) MAINTENANCE - SUB-COMPONENT NUMBERS

MAINTENANCE:

The mask's components, in particular the exhalation valve membrane and the valve seat, must be checked before use. A correctly functioning exhalation valve and membrane are just as important for the correct functioning of the mask as is the replacement of the filters at the correct time.

Check/Replacement of the exhalation valve membrane:

Open the cover to the exhalation valve. Holding the edge of the membrane, pull it carefully out. The valve seat must be clean and undamaged. If the membrane is deformed, split, or otherwise damaged, it must be replaced with a new original RSG-membrane (refer to maintenance intervals in table 3, page 8).

SUB-COMPONENT:

Sub-components which have a particular significance for safety are marked so that they can be identified - refer to table 4, page 8.

APPROVED INSTITUTION

System approval by: DEKRA EXAM GmbH, Adlerstrasse 29, D-45307 Essen, Germany, identification number 0158, has been involved in the construction phase of RSG HALF MASK

Production control according to Module C2 of PPE regulation (EU) 2016/425: DEKRA EXAM GmbH Zertifizierungsselle, Dinnendahlstrasse 9 • 44809 Bochum: Germany (CE 0158)

Download Declaration of Conformity @ www.rsgsafety.com/en/declaration-of-conformity

TABLE 1. CONSTRUCTION PARTS (Fig. 1):

Pos	Description	Pos	Description	Pos	Description
1	Head straps	5	Front piece	9	Exhalation valve
2	Strap buckle fram	6	Neck strap	10	Exhalation valve membrane
3	Buckle (quick release)	7	Inhalation valve membrane	11	Cover (Exhalation valve)
4	Buckle frame (quick release)	8	Mask body		

TABLES**(GB) 8****TABLE 2. SPARE PARTS:**

Pos. no. (Fig 1)	Description	Item no.	Pieces
1+6	Straps - Complete	30 19 03	1
7+10	Valve membrane set - Complete	30 19 03	1

TABLE 3. MAINTENANCE AND TEST:

Component	Service	SERVICE INTERVAL				
		After use	Half-yearly	yearly	Every 2 year	Every 6 year
Mask with membrane	Cleaning	•		• 1)		
	Disinfection	•		• 1)		
RSG HALF MASK	Function & leakage test	•	• 3)			
Exhalation valve membrane	Visual test	•	• 3)			
	Replacement	•			•	• 2)
Inhalation valve membrane	Visual test	•	• 3)			
	Replacement					•

1) Does not apply to masks in airtight packing.

2) Also applies to spare parts and masks in stock.

3) Only random sample tests apply if the masks are stored in airtight packing.


TABLE 4. COMPONENT NO:

RSG Mask Component	30 01 03 Large	30 01 02 Medium	30 01 01 Small	30 02 03 Large	30 02 02 Medium	30 02 01 Small
Mask body	30 19 05	30 19 05	30 19 05	30 19 06	30 19 06	30 19 06
Straps	30 19 03	30 19 03	30 19 03	30 19 04	30 19 04	30 19 04

RSG SAFETY B.V. 2018 (GB)


9 (GB)**MARKING**

Esimerkki:

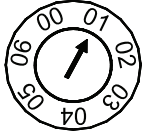**MARKING OF YEAR OF MANUFACTURE ON :****Mask****&****Straps**

In the example shown, the date of manufacture is 2001.

Esimerkki:

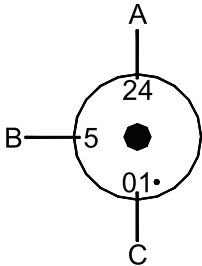**MARKING OF YEAR OF MANUFACTURE ON :****Exhalation valve membrane**

- A. Internal RSG-marking
- B. Internal RSG-marking
- C. The membrane's year of manufacture: The two-digit figure indicates the year of the start of production.

Each subsequent dot corresponds to one year. In the example shown, the date of manufacture is 2002.

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RSG SAFETY B.V. 2018 (FIN)